

DESIGNED FOR LEFT RADIAL ACCESS

Stay radial and stand tall with StandTall



Comfortably leverage the advantages of left radial artery access. An intuitively designed external vascular sheath extender, StandTall conveniently redirects workflow to the right side of the table. As a result, you can maintain hand dominance and posture, while reducing musculoskeletal stress and fatigue.

Making left right – for you

With StandTall, you get easy vascular access from the left radial artery. Use StandTall to stay radial to avoid the potential femoral procedural risks, so you and your patients can feel the benefits and realize the cost savings of radial procedures:

Physician benefits

- Decreased fatigue and musculoskeletal stress
- Increased site access and stability
- Improved workflow

Patient benefits


- Pronated arm position for greater comfort
- Faster reperfusion, recovery time
- Enables nondominated hand patient option in more than 90% of population
- Improved overall procedure satisfaction

Hospital benefits

- Reduced costs vs. femoral procedures
- Reduced post-operative patient management
- Reduced physician work-related absences due to MSK-stress injuries



With StandTall

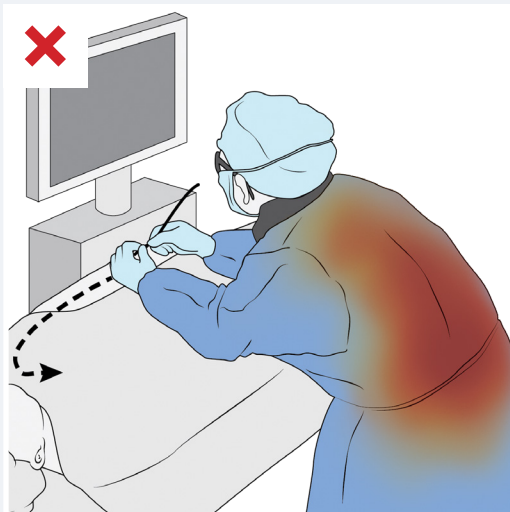

Without StandTall

Simple set up, ease of use

Setting up StandTall is easy and can be done in less than 2 minutes.

1. Review patient anatomy and conditions to establish StandTall length.
2. Flush StandTall and then attach to the vascular access sheath via the universal sheath adapter.
3. Position the adhesive clasp to secure StandTall in the optimal position and adhere it to the patient or sterile drape.
4. Shape StandTall into the desired position and secure it onto the adhesive clasp of the distal palm.
5. Pronate and position the left arm across the patient to the right side of the table and secure arm to prevent arm drift.

Rely on StandTall

For your convenience, the StandTall inner diameter measures 8.3 Fr and comes in 10 cm, 15 cm and 25 cm lengths to match individual patient anatomy and your preferred catheter system.



Radux Devices' mission is to protect healthcare professionals today for healthier lives tomorrow.

The company develops innovative medical devices that intuitively improve procedural performance and workflow, while delivering advantageous outcomes for physicians, nurses and hospitals.

Radux Devices, LLC
 11292 86th Ave N Suite 101
 Maple Grove, MN 55369
 P: 800.935.3137 F: 312.757.1279
 customerservice@raduxdevices.com

Please visit raduxdevices.com for supporting information.

TAKE RADIAL CONTROL

To learn more about the significant advantages StandTall provides, please contact **800-935-3137** or visit raduxdevices.com.