

# ACCESS MANAGEMENT FOR COMPLEX VASCULAR INTERVENTIONS

Maintain purchase, redirect workflow and stabilize vascular access during complex interventions with StandTall.

## Stand tall with StandTall

Comfortably leverage the advantages of attaining easier access for complex vascular interventions. An intuitively designed external vascular sheath extender, StandTall conveniently redirects workflow. As a result, you can maintain hand dominance and posture reducing musculoskeletal stress and fatigue.

### Physician and staff benefits


- Decreased fatigue and musculoskeletal stress
- Improved access site stability
- Improved wire and catheter management
- Expands access choices
- Redirect workflow
- Improved sterility
- Decreased scatter radiation

### Hospital benefits

- Reduced physician work-related absences due to musculoskeletal-stress injuries
- Improved sterility and table management reducing cost and improving patient outcomes



With StandTall

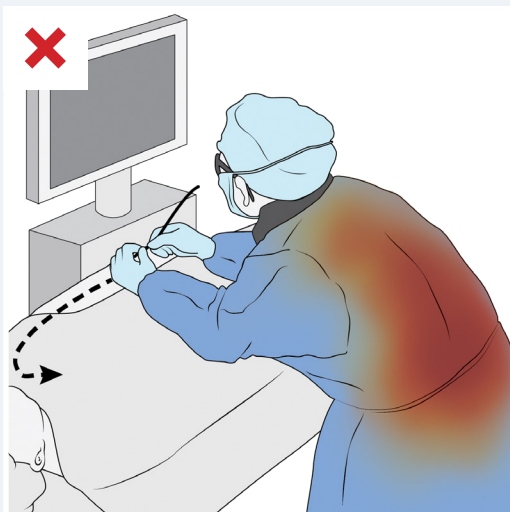

Without StandTall

## Simple set up, ease of use

Setting up StandTall is easy and can be done in less than 2 minutes.

1. Review patient anatomy and conditions to establish StandTall length.
2. Flush StandTall.
3. Attach StandTall to the vascular access sheath by gently applying force to penetrate sheath hemostasis seal with 2-3 clockwise rotations.
4. Position the adhesive horseshoe or radial clasp to secure StandTall in the optimal position and adhere it to the patient or sterile drape.
5. Shape StandTall and secure it onto the adhesive horseshoe or clasp.

## Rely on StandTall

For your convenience, the StandTall inner diameter measures 8.3 Fr and comes in 10cm, 15cm and 25cm lengths to match individual patient anatomy and your preferred catheter system.

### Improved access management

- Antegrade arterial puncture
- Arterial-venous fistula
- Carotid
- Pannus management
- Pedal
- Popliteal
- Radial
- Trans-lumbar



Radux Devices' mission is to protect healthcare professionals today for healthier lives tomorrow.

The company develops innovative medical devices that intuitively improve procedural performance and workflow, while delivering advantageous outcomes for physicians, nurses and hospitals.

Radux Devices, LLC  
 11292 86th Ave N Suite 101  
 Maple Grove, MN 55369  
 P: 800.935.3137 F: 312.757.1279  
 customerservice@raduxdevices.com

Please visit [raduxdevices.com](http://raduxdevices.com) for supporting information.

## TAKE CONTROL

To learn more about the significant advantages StandTall provides, please contact **800-935-3137** or visit [raduxdevices.com](http://raduxdevices.com).