

ACCESS MANAGEMENT FOR COMPLEX VASCULAR INTERVENTIONS

Maintain purchase, redirect workflow, & stabilize vascular access during complex interventions with StandTall™.

STAND TALL WITH STANDTALL™


Comfortably leverage the advantages of attaining easier access for complex vascular interventions. An intuitively designed external vascular sheath extender, StandTall™ conveniently redirects workflow. As a result, you can maintain hand dominance & posture, reducing musculoskeletal stress and fatigue.

Physician & Staff Benefits


- Decreases musculoskeletal (MSK) stress & fatigue
- Improves site access & stability
- Improves wire & catheter management
- Expands access choices
- Redirects workflow
- Improves sterility
- Decreases scatter radiation

Hospital Benefits

- Reduces physician work-related absences due to MSK-stress injuries
- Improves sterility & table management, reducing cost & improving patient outcomes

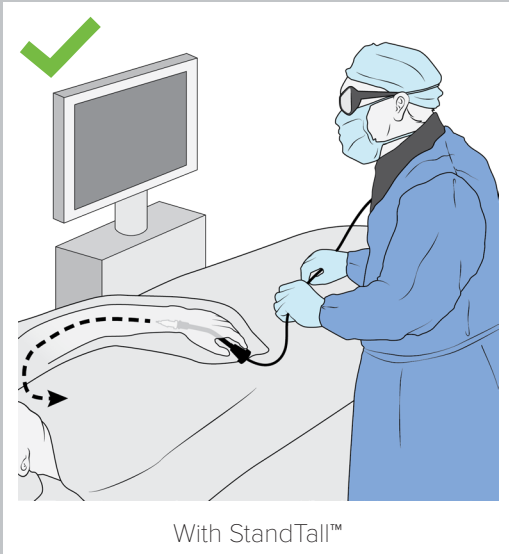

With StandTall™

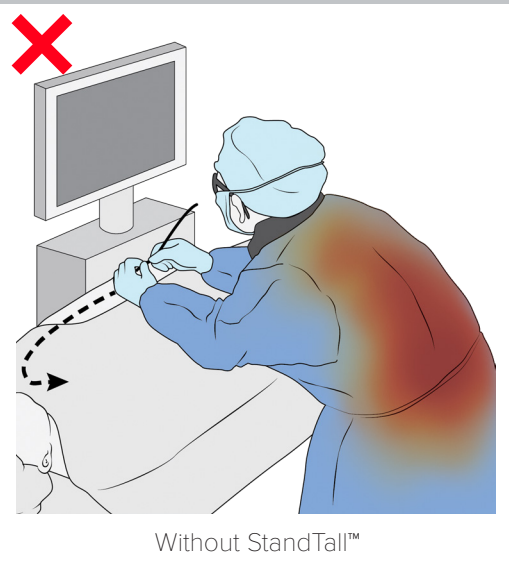

Without StandTall™

SIMPLE SET-UP, EASE OF USE

Setting up StandTall™ is easy & can be done in less than 2 minutes.

1. Review patient anatomy & conditions to establish StandTall™ length.
2. Flush StandTall™.
3. Attach StandTall™ to the vascular access sheath by gently applying force to penetrate sheath hemostasis seal with 2-3 clockwise rotations.
4. Position the adhesive horseshoe or radial clasp to secure StandTall™ in the optimal position & adhere it to the patient or sterile drape.
5. Shape StandTall™ & secure it onto the adhesive horseshoe or radial clasp.

RELY ON STANDTALL™

For your convenience, the StandTall™ inner diameter measures 8.3 Fr & comes in 10cm, 15cm, & 25cm lengths to match individual patient anatomy & your preferred catheter system.

Improved Access Management

- Antegrade arterial puncture
- Arterial-venous fistula
- Carotid
- Pannus management
- Pedal
- Popliteal
- Radial
- Trans-lumbar



Radux Devices' mission is to protect healthcare professionals today for healthier lives tomorrow.

The company develops innovative medical devices that intuitively improve procedural performance and workflow, while delivering advantageous outcomes for physicians, nurses, and hospitals.

Radux Devices, LLC
 11292 86th Ave N, Suite 101
 Maple Grove, MN 55369
 P: 800.935.3137 F: 312.757.1279
 customerservice@raduxdevices.com

Please visit www.raduxdevices.com for supporting information.

TAKE CONTROL

To learn more about the significant advantages StandTall™ provides, please contact 800-935-3137 or visit www.raduxdevices.com.

© 2023 Radux Devices, LLC. All Rights Reserved. StandTall™ is a Registered Trademark of Radux Devices, LLC. ML02-0002 Rev A